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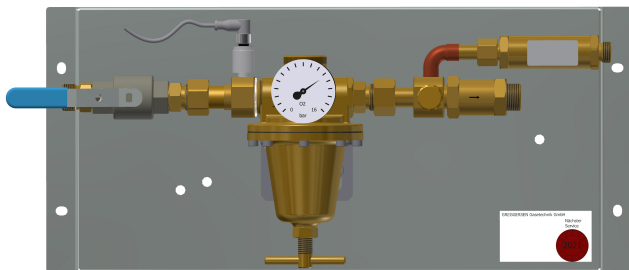
Reducer panel tank supply

Dear customers,

Thank you for purchasing our Greggersen product.

If you have any questions or require information, please contact our sales and support team.

+49-(0)40 739357-0, sales@greggersen.de

Please read the instructions for use thoroughly before using the medical device. Only use it once you have completely understood the application and function!

All serious incidents must be reported to Greggersen Gasetechnik GmbH and to the health authority responsible for the place of residence.

Symbols:

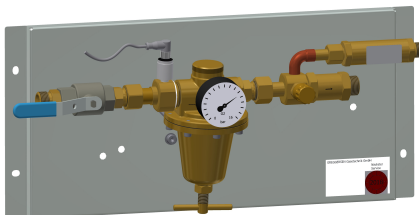
Date of manufacture
calender week year

Manufacturer

Keep free from
oil and grease


Use

The pressure reducer panel is used to reduce the pressure from a gas tank to a desired operating pressure. The tank pressure can be monitored via a sensor.

Device description

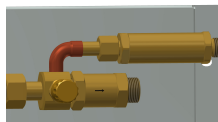The reducer panel has a pressure reducer and an associated safety valve. A ball valve can be used to isolate the system from the source. The system is suitable for oxygen and has an encapsulated 6 bar safety valve through which the gas can be routed to the outside. At the outlet, the pressure reducing panel has a non-return valve. In the low-pressure range, the system is equipped with a NIST coupling to enable emergency supply or sampling.

Encapsulated safety valve

Mounting

Fasten the reducer panel to the wall. Make the output connection to the supply grid. Lead outlets of the safety valves to the outside. Connect to a gas source. If necessary, connect the sensor to the monitoring electronics. After installation, flush the entire system.

Warnings

- During soldering, the lines must be purged with inert gas!
- Installation may only be carried out by authorized specialist personnel.
- Always keep all lines and connections free of oil and grease! **Danger of explosion !!!**

Service and maintenance

An inspection of the system is recommended once a year and may only be carried out by authorized specialist personnel. This includes checking the correct pressure conditions and the tightness of the piping system and valves. Service parts (seals, membranes, etc.) should be replaced every 5 years.



Service parts

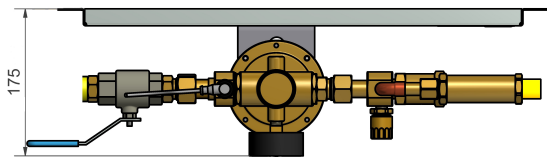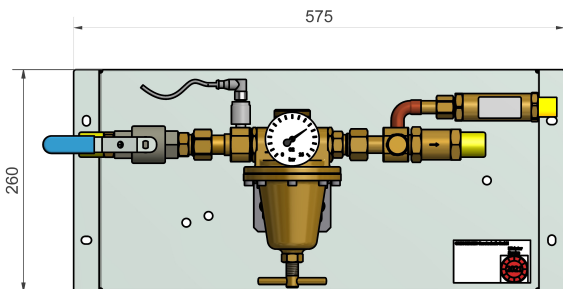Order number	Description
325348	Repair set tank supply „all in one“
906481	Repair set safety valve G3/8“ 6 bar

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GREGGERSEN Gasetechnik GmbH / Bodestr. 27-31 / 21031 Hamburg / Germany