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# Terminal unit Forano

**Dear customers,**

Thank you for purchasing this Greggersen product.  
If you have any questions or require information, please contact our sales and support team.

+49-(0)40 739 357-0; [sales@greggersen.de](mailto:sales@greggersen.de)

Please read the instructions for use thoroughly before using the medical device.  
Only use it once you have completely understood the application and function! All serious incidents must be reported to Greggersen Gasetechnik GmbH and to the health authority.

## Medical device in accordance with:

DIN EN ISO 9170-1  
(EU) 2017/745

## Connections in accordance with:

DIN 13260-2  
BS 5682  
NF S90-116

SS 875 24 30 (AGA)  
SN ENV 737-6:2005

## Symbols:

Consult the instructions for use

Keep free from oil and grease

Date of manufacture  
calendar week year

Temperature range

Medical device

Product contains lead


Manufacturer

Item number

## Labels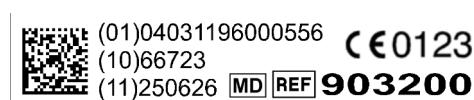

## Prolog

The Forano terminal unit from Greggersen Gasetechnik GmbH comply with the relevant provisions of Regulation (EU) 2017/745 of the European Parliament and Council and are classified as Class IIa active, non-invasive devices.

CE Marking: C 0123

Basis UDI: 4031196FORANOESTAS

## Intended use

Devices to distribute or supply medical gases or vacuum to equipment in diagnostic, therapy or surgery

## Functionality

The terminal units are intended to be connected to a gas supply system according to DIN EN ISO 7396-1 and to provide different types of compressed gases or vacuum. To withdraw the gases or vacuum, a plug must be connected to the terminal unit.

The plug is inserted into the terminal unit in the case of terminal units compliant with the DIN, BS, AGA and Carbamed standards. The plug locks automatically and simultaneously establishes a connection to the gas or vacuum source by opening the sealing cone in the rear part. Once the application is finished, the plug can be released from the terminal unit by pressing the actuator, disconnecting the gas connection at the same time. In the case of the DIN terminal units, the plug can be brought into the parking position by easy connection and disconnection. Here the plug is held in the terminal unit without a gas connection being established.

With the tapping points of the NF standard, the plug must be locked with the union nut by turning it clockwise after it has been plugged into the tapping point. To unlock, the plug must be pressed into the tapping point and at the same time the union nut must be turned anti-clockwise.

To permanently stop operation of the terminal units, the sealing cone must be pressed into the terminal unit and screwed down. This seals the outlet point gas-tight and the front part can then be dismantled.

In the DIN, AGA, BS and Carbamed versions, all sealing elements are located in a replaceable unit, the cartridge.

## Warnings

- Prior to the use of the device, the user has to be familiar with the product and the instructions for use and must have understood them
- The terminal unit must be checked for external damage, contamination and the right gas type lettering before each use. It must be ensured that the outlet unit is correctly aligned and that the labelling on the terminal unit is horizontal.
- When cleaning, make sure that no liquids enter the terminal unit. After cleaning, the terminal unit must be dried completely.


- Oxygen is a highly oxidizing gas. Always keep all connections free of oil and grease! No smoking and no fire near oxygen equipment! Fire hazard!
- After installation or maintenance of the terminal unit, an acceptance check according to DIN EN ISO 7396-1 must be performed.
- Only use the terminal unit with plugs of the respective gas standard.
- Check the gas connection after connecting a DIN plug. The plug can be in the parking position.
- All serious incidents must be reported to Greggersen Gasetechnik GmbH and to the health authority responsible for the place of residence.
- Do not use the terminal unit in areas with high magnetic or electromagnetic fields (MRI).
- Our products are partly made of brass with a lead content of more than 0.1%. As an alloy component, lead is firmly bound in the material so that no exposure is to be expected.
- The visible area of the outlet must not be covered with self-adhesive materials at any time.

## Device description


### Terminal unit DIN, DIN, BS, SN (Carbamed) SS (AGA)

| No. | Description     |
|-----|-----------------|
| 1   | Base block      |
| 2   | Mounting screws |
| 3   | Sealing cone    |
| 4   | Cartridge       |
| 5   | Front part      |
| 6   | Pressure ring   |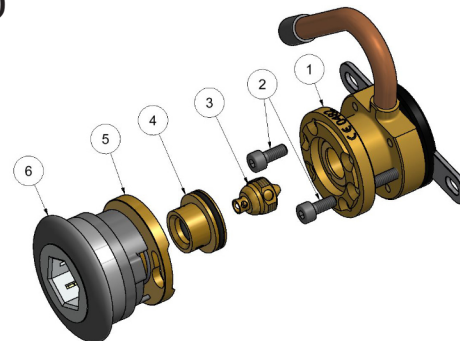

### Terminal unit NF

| No. | Description     |
|-----|-----------------|
| 1   | Base block      |
| 2   | Mounting screws |
| 3   | Sealing cone    |
| 4   | O-ring          |
| 5   | Front part      |
| 6   | Pressure ring   |
| 7   | Valve insert    |

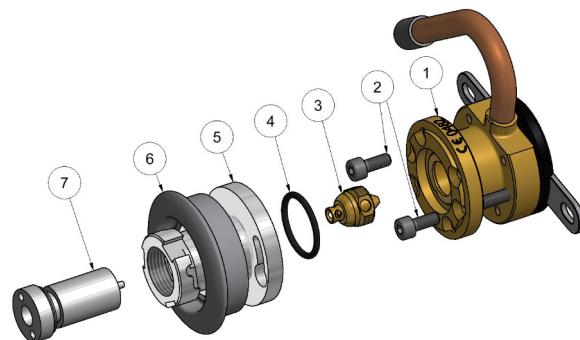## Patient target group

The terminal units are used to provide medical gases or vacuum.

The application of medical gases is determined by the medical personnel operating the terminal unit. Medical gases can be used for different purposes. No specific determination of the disease or condition, clinical form, stage, severity, symptoms or aspects to be treated/diagnosed is therefore possible.

## Intended users

The medical device is intended for clinical use by medically qualified personnel (e.g. therapists, care personnel, doctors).

## Operation

Before starting operation, an external visual inspection and a functional test have to be carried out each time.

During the external visual inspection, the terminal unit is checked for obvious damage or contamination. During the function test, the pressure ring has to be pressed to check the ease of movement.

To establish a gas or vacuum connection, a plug must be connected in the terminal unit. Please note that a parking position is also possible for DIN standard terminal units. To bring a plug into the parking position, it is only inserted up to the first locking point. The gas connection has not yet been established in this position. To establish a gas connection, connect the plug up to the second locking point. NF standard plugs must be secured using the bayonet lock.

To release a plug, the actuator of the terminal unit must be pressed to pull the plug out of the terminal unit. In the case of DIN terminal unit, the plug can be moved to the parking position by simply pulling it beyond the first locking point. In case of NF standard terminal units, the plug has to be pressed slightly into the terminal unit and the bayonet lock released.

## Cleaning

The terminal units can be wet cleaned externally with commercially available disinfectants and cleaning agents. When cleaning, make sure that no liquid enters the terminal unit. The terminal unit must be completely dry prior to subsequent use. The hygiene plan of the healthcare facility must be observed.

## Installation

- Attach the base block depending on the type of installation.
- Solder the base block to the piping system.

Caution!

During soldering, the lines must be flushed with inert gas (N<sub>2</sub>)! To do this, connect the line to a source of inert gas and press the locking cone down until the interior of the terminal unit is sufficiently flushed.


- Alternatively for terminal units with NIST connection: Connect the coupling parts and tighten the union nut by hand.
- Depending on the type of installation, put on the front panel or cover ring
- Place the front part of the terminal unit and screw it on.
- Click the actuator on.
- After installing or replacing the cartridge, the terminal unit must be activated once with a plug. This aligns the closing cone and prevents leaks.

- Establish the gas connection.

#### Caution!

After installation, an acceptance test must be conducted in accordance with DIN EN ISO 7396-1. The required pressure and leak tests must be performed with the front section in place. Alternatively, the closing cone can be screwed down (see Maintenance and Inspection) and the base block can then be sealed with a blind flange. When closing the tapping point by screwing down the closing cone, as described in the Maintenance and Inspection section, 100% leak tightness cannot be guaranteed! Installation may only be carried out by trained personnel.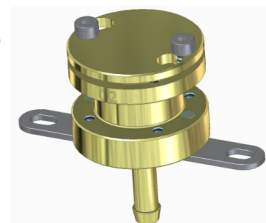

### Disposal

There are no special disposal requirements for this device. The device can be disposed of in household waste. Installation may only be carried out by properly trained and qualified personnel.

### Maintenance and inspection

A function and tightness check according to DIN EN ISO 9170-1 must be carried out at least once a year. The tightness of the locking system of the tapping points of the country standards according to DIN, BS and SS must be checked using the leak test plug from the test kit. Maintenance must be carried out at least every 6 years, whereby the cartridge has to be replaced. In case of intensive use, more frequent replacement of wearing parts may be necessary. Wear can lead to leaks and thus to audible hissing noises from the terminal unit. To replace the wearing parts, the sealing cone is screwed into the terminal unit with a 3 mm Allen key, which interrupts the gas flow. The actuator can then be pulled off and the front part removed by loosening the fixing screws. The cartridge is now accessible and can be changed. The front part then has to be reassembled and the gas flow opened. At the NF standard tapping points, the valve insert must be changed, for which the gas supply must be interrupted. On completion of maintenance work, an acceptance test according to DIN EN ISO 7396-1 must be carried out. Maintenance may only be carried out by properly trained and qualified personnel.

### Technical data

|                     |                                                                                                                                                                                                                                                                                                                                                 |
|---------------------|-------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|
| Operating pressure: | Compressed gases 400...500 kPa, max. 1000 kPa<br>vacuum $\leq$ -40 kPa                                                                                                                                                                                                                                                                          |
| Test pressure:      | Compressed gases max. 1500 kPa;<br>vacuum 500 kPa                                                                                                                                                                                                                                                                                               |
| Gas types:          | Oxygen (O <sub>2</sub> ), compressed air (AIR / MA), vacuum (VAC),<br>nitrogen (N <sub>2</sub> ), nitrous oxide (N <sub>2</sub> O), carbon dioxide (CO <sub>2</sub> ),<br>nitrogen 800 (N <sub>2</sub> -800), compressed air 800 (AIR-800 / SA),<br>argon (Ar), oxygen 93 (O <sub>2</sub> 93),<br>terminal unit for excess anesthetic gas (UTL) |
| Standards:          | DIN, BS, SS (AGA), NF, SN (Carbamed)                                                                                                                                                                                                                                                                                                            |

|                          |                                                                                                                                                                                                      |
|--------------------------|------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|
| Installation types:      | Concealed / hollow wall, surface-mounted, rail mounting, ceiling supply units, pipe at rear, straight pipe, hose connection at top, DVE-NIST, concealed hose connector, Concealed / hollow wall rear |
| Gas type identification: | colour-neutral or according to ISO32                                                                                                                                                                 |
| Operating temperature:   | 0°C to +40°C                                                                                                                                                                                         |
| Storage temperature:     | -20°C to +60°C                                                                                                                                                                                       |
| Expected lifespan:       | 10 years                                                                                                                                                                                             |
| Dimensions:              | see page 87                                                                                                                                                                                          |

### Maintenance parts

| No.    | Description                              |
|--------|------------------------------------------|
| 327463 | Forano Cartridge DIN                     |
| 327468 | Forano Cartridge AGA                     |
| 327447 | Forano Cartridge Carba                   |
| 325332 | Forano Cartridge BS                      |
| 326436 | Forano Cartridge AGA Vac                 |
| 326457 | Cartridge conversion kit DIN - Forano II |
| 103401 | NF socket valve insert compressed gases  |
| 103402 | NF socket valve insert vacuum            |
| 104785 | NF socket valve insert AIR800            |

### Accessories

| No.    | Description                                             |
|--------|---------------------------------------------------------|
| 903258 | Forano front plate packing unit = 1 piece               |
| 903259 | Forano front plate packing unit = 6 pieces              |
| 903309 | Forano "not operative" sticker packing unit = 25 pieces |
| 903380 | Forano child safety lock AIR or O2                      |
| 903381 | Forano child safety lock CO2 / N2O / VAC                |
| 903308 | Key for Forano child safety lock                        |
| 903302 | Forano actuator disassembly tool                        |
| 903301 | Forano blind flange for pressure test                   |
| 903300 | Forano height adjustment assembly tool                  |
| 903303 | Forano mounting aid for placing 4 terminal units        |
| 903304 | Spanner for sealing cone retaining screw                |
| 903305 | Forano sealing cone Allen key                           |
| 326380 | NF valve insert key compressed gases                    |
| 326381 | NF valve insert key VAC/AIR800                          |
| 902114 | Test kit terminal units acc. to 7396-1/2 DIN            |
| 902128 | Test kit terminal units acc. to 7396-1/2 BS             |
| 902129 | Test kit terminal units acc. to 7396-1/2 AGA            |

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